
## Adjustable Hernia Belt InnovaGoods® -

### InnovaGoods®

#### User's Manual

- InnovaGoods® over your underwear (never place it directly in contact with the skin).
- Put the belt around your waist and ensure it feels well secured (1).
- Pass the fastening strap through the loop supporting the pad (C) to adjust it. Then adjust the strap until you find the most comfortable position (2).
- Put the removable pad (D) inside the inner part of the support and place it where you need it. When you find the right position for it, secure it with the Velcro (3).
- Once the pad is in place, secure the lower fastening strap with the Velcro at the back to prevent the hernia belt and the pad from moving out of place (4).
- Once in place, if you need to readjust the belt during the day, carry out the above steps again. It is important not to press too hard in order to allow good circulation and to find the optimum position that ensures your comfort (5).

InnovaGoods®  
www.innovagoods.com

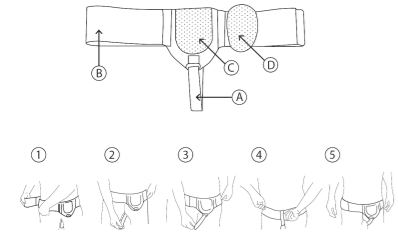### ENGLISH

#### WARNINGS

**IMPORTANT: KEEP THE INSTRUCTIONS FOR FUTURE REFERENCE.** Before using the Adjustable Hernia Belt InnovaGoods®, read the user manual carefully and follow the instructions closely. Check that the product contains all the components as described in the placing procedure. Regularly check its condition. Inappropriate use could cause serious health problems. Do not expose the product to situations that may cause it to catch fire. If this occurs, remove it quickly and use appropriate measures to extinguish it. Use of the Adjustable Hernia Belt InnovaGoods® will not cure or treat hernias: this product relieves the pain in order to give the people with hernias, or who are in the post-operation stage, greater comfort in their day-to-day life. If you experience discomfort such as chafing, irritation or swelling, remove the product immediately and seek medical advice. If the pain caused by the hernia is not reduced, loosen the hernia belt and re-adjust it until it feels comfortable. If the pain persists, stop using the product immediately and consult your doctor. Use only on healthy skin without open wounds. Read the instructions carefully before first use. Keep the manual for further queries.

#### CONTENTS

- 1 Adjustable Hernia Belt InnovaGoods®
- 1 removable pad

#### INSTRUCTIONS

- Before putting on the Adjustable Hernia Belt InnovaGoods®, ensure that the lower fastening strap (A) and the belt (B) are unfastened.
- Place the Adjustable Hernia Belt

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